

INSTRUCTIONS FOR USING WHITE LIQUID TO COVER DISCOLORED TEETH



white liquid

High strength zirconia block is required for coping. We(DMAX HQ) recommend Natura Eco or Alpha HT for coping.

01 Before use, shake the white liquid sufficiently and take it with small amounts using an eye dropper. White liquid has a reddish color due to the influence of the pigment.

02 Apply the white liquid only once on the inner surface of coping or full zirconia. Applying only once will give you a sufficient effect.

In the case of full zirconia, if an excessive amount of the white liquid is applied to the inner surface of the cut, it can also be seen on the labial side. So, pay special attention when applying the white liquid to the inner surface of the cut in the case of full zirconia.



03 Apply just once on the inner surface. If you apply an excessive amount of white liquid to the margin area, it may invade the labial area due to carelessness. Special attention is required for this job. In some cases, it is possible to apply except for the margin area.



04 Dry for 15 minutes at 200°C.



05 Keep regular sintering schedule. (It is essential to keep for 2 hours at the maximum temperature of 1530°C). After sintering, you can check the opaque effect of the part where the white liquid is applied.